

CYBERTECH PATHWAY LS (LUMBAR STABILIZER)

Code Approved L0627

The All NEW Cybertech Pathway LS (Lumbar Stabilizer)

This LSO allows the practitioner various Pre and Post operative management options. It allows for a progression of rehab adaptability with removable/replaceable anterior and posterior panels.

- Molded posterior panel contoured to the natural shape of the lumbar anatomy.
- A unique low profile removable anterior panel, pocketed into the belt for greater patient comfort and compliance.
- Hand straps for patients with arthritic hand involvement.
- Quick, one-hand 5:1 ratio, low friction Flexible Pulley System provides powerful, smooth and easily-controlled compression. support comfort for standing and sitting.

Indications:

Lumbar mechanical back pain, spinal stenosis, trauma, spondylolysis, spondylolisthesis, degenerative disc, herniated disc, bulging disc, facet syndrome, post operative stabilization protocol following: laminotomy, foraminotomy, IDET procedures, anterior laproscopic fusion, posterior lateral fusion, and multi-level decompression



Anatomically
Contoured
Posterior Panel

Hand Straps for
Arthritic Hand
Involvement



Low Profile Pocketed Anterior Panel

Cybertech[®]
MEDICAL

Cybertech Pathway LS (Lumbar Stabilizer)

Clinical Guidelines:

CYBERTECH Medical braces are medical products registered with the FDA as a "medical device". CYBERTECH Medical does not prescribe braces or determine the appropriateness of any particular bracing methodology for the individual. Medical advice should be sought on the appropriateness and the condition under which a given brace should be worn."

A general guide to clinical indications for each CYBERTECH Medical spinal orthosis include, but are not limited to, the following:

Indications:

Lumbar mechanical back pain, spinal stenosis, trauma, spondylolysis, spondylolisthesis, degenerative disc, herniated disc, bulging disc, facet syndrome, post operative stabilization protocol following: laminotomy, foraminotomy, IDET procedures, anterior laproscopic fusion, posterior lateral fusion, and multi-level decompression

Fitting Specifications:

Size	Circ.	Posterior Panel Ht.	Anterior Panel Ht.	Item#
Small	25"-30"	9 ¾"	7"	60-3212-1
Medium	30"-35"	9 ¾"	7"	60-3212-2
Large	35"-40"	9 ¾"	7"	60-3212-3
X-Large	40"-45"	9 ¾"	7"	60-3212-4
XX-Large	45"-50"	9 ¾"	7"	60-3212-5
XXX-Large	50"-55"	9 ¾"	7"	60-3212-6

U.S. Patent No. RE35,940



Fitting Instructions:

- Fasten the handle near the pulley unit. The handle is always on the left.
- Stand holding the soft side panel apart, centering the back panel on the spine.
- Wrap the left side to the front positioning the front panel.
- Wrap the right side on top of the left panel to close and attach in front. Adjust the front or back panels as needed.
- Pull the handle out and away from the power unit to desired compression and support. Place the handle on the fabric fastener band securely.
- Place the handle on the fabric fastener band securely.

Tips:

- Be sure to start the handle near the pulley unit.
- Extend or pull out the brace before application.
- Adjust tension for sitting or standing, experiment to meet your needs.
- Hand wash in warm water, rinse well, towel and air dry.

Posterior and Anterior Panels:

- Dynamic posterior panel conforms to each patient's individual body structure for improved comfort and support.
- Posterior panel has 2 height adjustment positions for additional customization.



1815 Wright Ave., La Verne, CA 91750
 Ph: 1.800.220.4224 Fax: 1.909.447.7058
www.cybertechmedical.com

© 2012 Bio Cybernetics International, Inc. All rights reserved. Rev. 07/12

CE
 Authorized Rep.
 RSQA / RSQR Ltd.
www.rsqa.co.uk
info@rsqa.co.uk